




As of 22 May 2024



USER INFORMATION

	<p>Comply to PPE Regulation 2016/425 Cat I (For Minimal Risk Only)</p> <p><i>Note: Please refer to the Declaration of Conformity for more details</i></p>
---	---

Product: Simplistic Powderfree Latex Gloves

Catalog No./ Product reference: SLPF01-XS, SLPF02-S, SLPF03-M, SLPF04-L, SLPF05-XL

Sizes available: XS, S, M, L, XL

Colour: White

Intended Use:

A patient examination glove is a medical device intended for a medical purpose that is worn on the examiner's hand or finger to prevent contamination between the patient and examiner. Examination glove is intended for medical activities except for surgery.

Before usage, inspect the gloves for any defects or imperfections.

Storage and transport: When not in use, store the product in a well-ventilated area away from extremes of temperature

For single use only, do not littering.

Check for damage before use, do not use damaged gloves

Donning:

1. Remove all hand and wrist jewelry, and wash the hands before donning.
2. Place the gloves on the prepared work surface.
3. The user puts a glove on his/her dominant hand by grabbing it with the other hand, remembering to only touch the inside of the gloves, and slipping it over the dominant hand until it reaches the final level.
4. The wearer uses the gloved dominant hand to slip the other glove onto the non-dominant hand.
5. Once both gloves are on, the users can touch the outside of the gloves to ensure a proper fit









**Doffing:**

1. Using the dominant hand, users start by grabbing the outside of the glove on the non-dominant hand on the palm side near the cuff.
2. Pull the glove off the non-dominant hand and place it in the gloved hand, balling it up.
3. Slip two fingers under the cuff of the other hand glove and carefully peel it off the hand without touching the wrist, turning the remaining glove inside out as it is removed and in turn encasing the first glove.
4. The gloves can be disposed.

The DOC (Declaration of Conformity) will be shown on the website: www.mymedikal.com.tr/documents

Product manufactured by: MY TICARET VE MEDİKAL A.Ş.

**Other Common Graphical Symbols:**

SYMBOLS	TITLE
	Manufacturer
	Date of Manufacture
	Use-by date/ Expiration Date
	Batch Code/ Lot Number
	Keep away from sunlight
	Keep dry





MY Medikal

	Do not re-use
	Non-sterile

MY TİCARET VE
MEDİKAL ANONİM ŞİRKETİ
Ömerli: Mh.General Şükrü Koralı Cad
No: 33 Arnavutköy/İSTANBUL
B.ÇEKMECE V.D.6260404605
İl:0212 436 20 64 Fax:0212 436 20 65
www.mymedikal.com

