




As of 02 Jan 2025



#### USER INFORMATION

	<p>Comply with PPE Regulation 2016/425 Cat I-For Minimal Risk Only</p> <p><i>Note: Please refer to the Declaration of Conformity for more details</i></p>
---	---

#### Product references:

**Mumu Powderfree Latex Gloves**

MLPF02

**Sizes available:** XS, S, M, L, XL

#### Intended Use:

The Latex Examination Gloves is a disposable device intended for medical and dental purposes that is worn on the examiner's hand to prevent contamination between patient and examiner.

“Before usage, inspect the gloves for any defect or imperfections.”

**Storage and transport:** When not in use, store the product in a well-ventilated area away from extremes of temperature

For single use only, do not littering.

Check for damage before use, do not use damaged gloves

#### Donning:

1. Remove all hand and wrist jewelry, and wash the hands before donning.
2. Place the gloves on the prepared work surface.
3. The user puts a glove on his/her dominant hand by grabbing it with the other hand, remembering to only touch the inside of the gloves, and slipping it over the dominant hand until it reaches the final level.
4. The wearer uses the gloved dominant hand to slip the other glove onto the non-dominant hand.
5. Once both gloves are on, the users can touch the outside of the gloves to ensure a proper fit






**Doffing:**

1. Using the dominant hand, users start by grabbing the outside of the glove on the non-dominant hand on the palm side near the cuff.
2. Pull the glove off the non-dominant hand and place it in the gloved hand, balling it up.
3. Slip two fingers under the cuff of the other hand glove and carefully peel it off the hand without touching the wrist, turning the remaining glove inside out as it is removed and in turn encasing the first glove.
4. The gloves can be disposed.





The DOC (declaration of conformity) will be shown on website: [www.mymedikal.com.tr](http://www.mymedikal.com.tr)

Product manufactured by: MY TICARET VE MEDİKAL A.Ş.



	This product also complies with Medical Device Regulation (EU) 2017/745 under the Class I category.
---	---

**Other Common Graphical Symbols:**

SYMBOLS	TITLE
	Manufacturer
	Date of Manufacture
	Use-by date/ Expiration Date
	Batch Code/ Lot Number





# MY Medikal

	Keep away from sunlight
	Keep dry
	Do not re-use
	Non-sterile
<b>EN455</b>	European Standard

MY TİCARET VE  
MEDİKAL ANONİM ŞİRKETİ  
Ömerli: Mh.General Şükrü Koralı Cad.  
No: 33 Arnavutköy/İSTANBUL  
B.ÇEKMECE V.D.6260404505  
İl:0212 436 20 64 Fax:0212 436 20 65  
www.mymedikal.com

