




As of 09 Aug 2024



USER INFORMATION

	Comply with PPE Regulation 2016/425 Category I- For Minimal Risk Only
---	---

The product complies with PPE Regulation 2016/425 Category I- For Minimal Risk Only which intends to protect users against minimal risks and doesn't provide protection against severe hazards. Due to that, the conformity assessment procedure doesn't include Notified body testing and evaluation. Thus, the manufacturers can self-declare compliance with the respective legislation.

Mumu Basic Latex Powdered Disposable Gloves

Product references:

MBLPPST01-XS, MBLPPST02-S, MBLPPST03-M, MBLPPST04-L, MBLPPST05-XL

Product Group Reference: LX01

Sizes available: XS, S, M, L, XL

Intended Use:

A disposable glove is intended for general use to provide safety for the user. This is used for minimal risk only.

The DOC (Declaration of Conformity) will be shown on website: www.mymedikal.com.tr

Product manufactured by: MY TICARET VE MEDİKAL A.Ş.





CAUTIONS AND RESTRICTION

1. Keep in dry conditions and avoid sunlight or other ozone sources.
2. For single-use only.
3. For latex gloves: This product contains natural latex which may cause allergic reactions.

DISPOSAL METHOD

Throw it away in the special rubbish can after using it.

INSTRUCTIONS FOR USE

1. Always wash hands before putting on gloves and each time you change to a new pair.
2. Take the gloves out from their original box.
3. Select a glove appropriate for the task and in your size. Inform yourself prior to the application.
4. Touch only a restricted surface of the glove corresponding to the wrist (at the top edge of the cuff)
5. Don the first glove by inserting your hand into the opening and pulling it over by applying a soft pull with the second hand.
6. Take the second glove with a bare hand and touch only a restricted surface of the glove corresponding to the wrist.
7. To avoid touching the skin of the forearm with the gloved hand, turn the external surface of the glove to be donned on the folded fingers of the gloved hand so there is a good hold on it. Thus, permitting to glove the second hand by pulling softly with the first gloved hand.
8. Once gloved, hands should not touch anything else that is not defined by indications and conditions for glove use.
9. Change gloves before beginning a different task to avoid cross-contamination.
10. Pinch one glove at the wrist level to remove it, without touching the skin of the forearm, and peel it away from the hand, thus, allowing the glove to turn inside out.
11. Hold the removed glove in the gloved hand and slide the fingers of the ungloved hand inside between the glove and the wrist of the gloved hand. Remove the second glove by rolling it down the hand and fold the first glove into the outside palm area of the second glove. If done correctly, the user holds a pack of the outer glove turned inside and out and the contaminated surface of both gloves is within the package.
12. Discharge the removed gloves and wash hands thoroughly according to the respective hygiene standards. Be aware to remove any powder residuals from the hands before touching any surfaces, as the powder can carry natural rubber latex proteins.
13. Not intended to be used as a chemical barrier. Check the original dispenser instructions for details.





MY Medikal

Other Common Graphical Symbols:

SYMBOLS	TITLE
	Manufacturer
	Date of Manufacture
	Use-by date/ Expiration Date
	Batch Code/ Lot Number
	Keep away from sunlight
	Keep dry
	Do not re-use
	Non-sterile

MY TİCARET VE
MEDİKAL ANONİM ŞİRKETİ
Ömerli: Mh.General Şükrü Koralı Cad
No: 33 Arnavutköy/İSTANBUL
B.ÇEKMECE V.D.6260404505
İl:0212 436 20 64 Fax:0212 436 20 65
www.mymedikal.com

